



Trionic Walker Backrest – Assembly Instruction



The ergonomic and padded backrest, made from water-repellent and durable 600D polyester, fits the rollators Walker 9er, 12er and 14er. It offers a comfortable and safe support when you are seated.

The length/depth of the backrest is adjustable.

① Unscrew.

Remove the screws and nuts from the two clamps.

② Attach the backrest.

Open the clamps a little and attach them to the grip tubes.

IMPORTANT: Make sure that the "TRiONiC" logotype is positioned correctly, as the backrest is angled.



③ All the way up.

Push the two clamps to the very top. You may have to loosen the screw on the brake lever in order to move it rearwards a bit.





④ fasten the backrest.

With a 4 mm allen key, fasten both clamps with the screws and nuts. The straps of the backrest should point forward at the point of attachment.

⑤ This photo shows the correct assembly.



Before unfolding the rollator.

To unfold the rollator, angle the backrest upwards.

If necessary.

If you find the backrest being too short/too long, you can adjust the length. Remove the two screws and nuts and then assemble the backrest again.

