




## **Manual: Replacement of Trionic Walker Backrest Strap**



- ① Unscrew the screw that holds the strap with a 3mm allen key.  
Repeat on the opposite side.



- ② Make sure that you recover the screw, the nut and the tubular spacer.

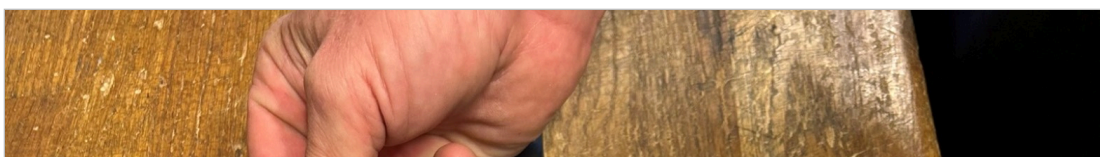



## **Manual: Replacement of Trionic Walker Backrest Strap**



- ⑤ Insert the new replacement strap into the padding.

**IMPORTANT:** Make sure to align the end of the strap with the TRIONIC logo as shown in the photo.



- ⑥ Insert the tubular spacer into the hole at the end of the strap.

