



Walker Transport Bag - Packing Instructions


The Walker transport bag consists of a main transport bag and a smaller storage-/carry bag. The transport bag fits all Walker models.

Fits all Trionic Walker models.




① Open the main bag

Unzip the storage bag and unpack the main transport bag.

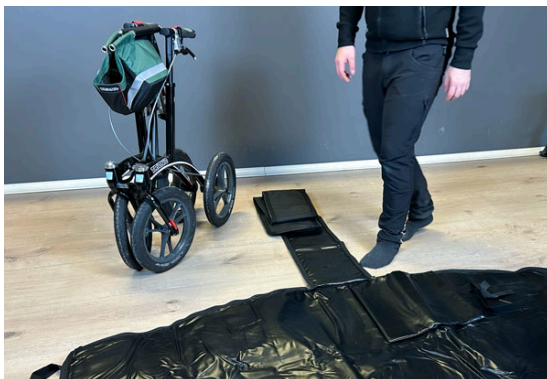

② Spread the main bag.

Unzip the full length of the main bag's zipper and unfold the flaps of the bag on the floor.

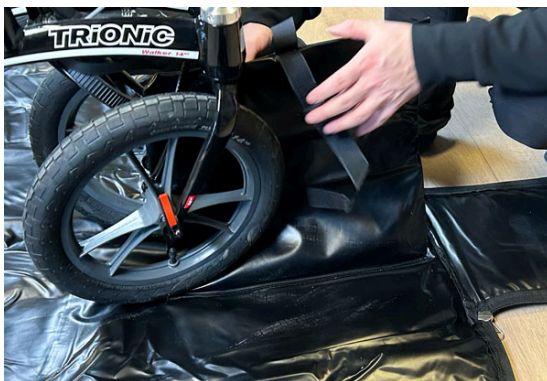

③ Place the walker on the bag.

Place the Walker on the center section of the main transport bag. The main bag has an integrated padded protection flap with a Velcro strap. Pull the padded protection flap upwards and place it between the front wheel forks in order to protect the paintwork.



- ④ Secure the folded frame with the integrated Velcro strap.



- ⑤ Fold the center flap over the Walker.



- ⑥ Lift the side flaps and close the main zipper.



- ⑦ This photo shows the Walker as correctly packed.

Packing instructions



Scan the QR-code with your smart phone or device to view video instructions.